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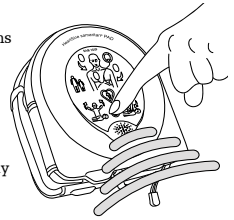
EN Please store this supplement with your user manual/user handbook and AED.

As part of the monthly maintenance check, HeartSine Technologies recommends that the user performs the following check once every three months. This can be done quickly, without removing the AED from its case.

- Press the On/Off button to turn on the AED
- Listen for, but do not follow, the voice prompts to ensure you can hear the prompts
 - If you do not hear a prompt, contact your authorized distributor or HeartSine Technologies directly
 - If you hear the prompt "Adult patient" and/or "Call for medical assistance," no further action is needed
- Press the On/Off button again to turn off the AED and verify that the status indicator is flashing green
 - If you hear a warning message or see a red flashing status indicator, consult Troubleshooting in the user manual/user handbook
- You can record the AED serial number (SN) and year (YYYY), month (MM) and day (DD) for each of these maintenance checks in the log provided at the end of this supplement for your convenience

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Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: HeartSine, samaritan, Stryker. All other trademarks are trademarks of their respective owners or holders.
Copyright © 2024 Stryker.
HeartSine Technologies, Ltd.